

Pill Cutter and Grinder

Item Catalog Number: 00614

Splitting pills in half, or even quarters, is relatively easy with the Pill Cutter/Grinder. Designed with a sturdy and wide non-slip base, this tool allows you to cut large and small pills. The device is also equipped to grind. This function should be used to convert large tablets into powder for administration to those who have difficulty swallowing. The machine also conveniently stores your pills.

The machine also conveniently stores your pills.

Dosage and Use

- To cut: Open lid. Insert pill in the center of the diamond shape. Close lid while pressing down. Remove pieces.
- To grind: Unscrew the turquoise colored part. Insert pill to be ground. Screw back in to close completely. To grind to a fine powder, repeat several times.
- To store: Lift the burgundy colored portion by the handles on either side. Insert pills or pill portions into the compartment.
- Always clean compartments well with a brush after each use.

Caution: Tablets that are enterically coated should not be split.

All Contents Copyright © 1995-2009 Life Extension Foundation All rights reserved.

LifeExtension®

These statements have not been evaluated by the FDA. These products are not intended to diagnose, treat, cure or prevent any disease. The information provided on this site is for informational purposes only and is not intended as a substitute for advice from your physician or other health care professional or any information contained on or in any product label or packaging. You should not use the information on this site for diagnosis or treatment of any health problem or for prescription of any medication or other treatment. You should consult with a healthcare professional before starting any diet, exercise or supplementation program, before taking any medication, or if you have or suspect you might have a health problem. You should not stop taking any medication without first consulting your physician.